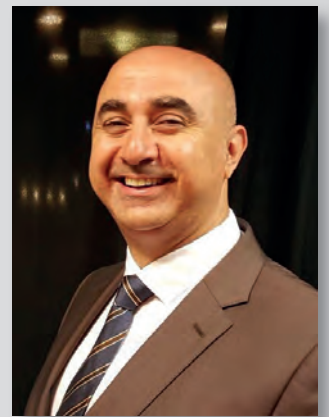
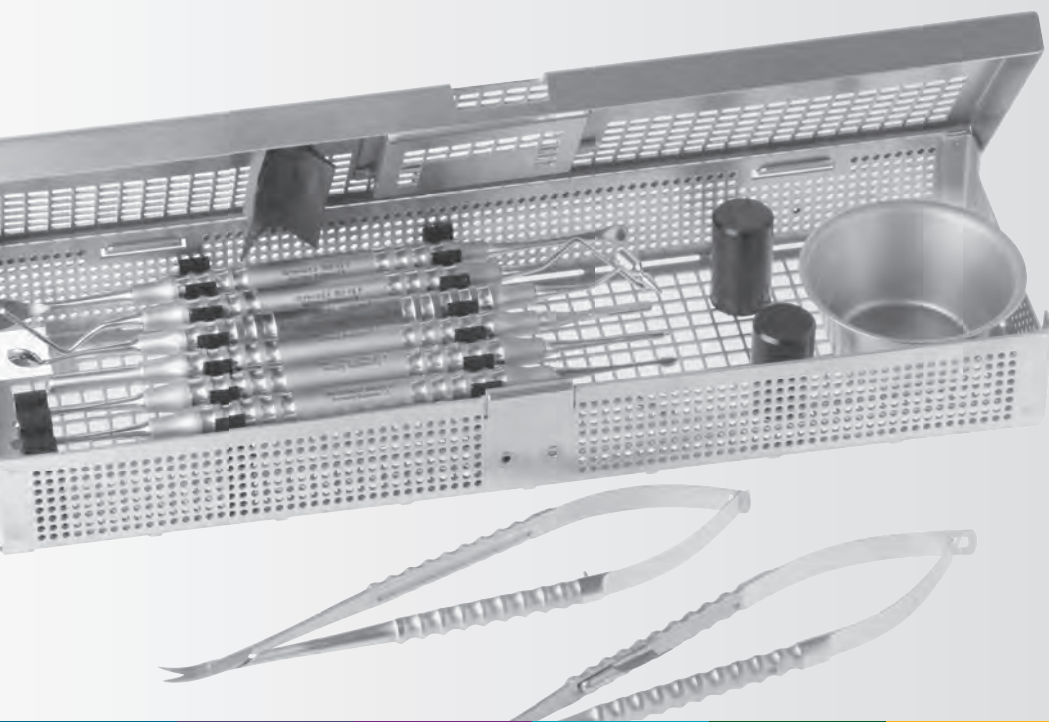
*... makes the difference*

**RECOMMENDATION OF  
DR. BASSAM RABIE**

see it feel it test it

devemed  
... makes the difference

## Dr. Bassam F. Rabie

Dr. Bassam F. Rabie graduated from the Faculty of Dental Medicine, Cairo University in 1992, where he received his BDS. He was appointed a teaching position in the Faculty of the Dental School for 3 years. In 1997 Dr. Bassam finished his Implant Prosthodontic Residency Speciality Program from the School of Dental Medicine, Pittsburgh, Pennsylvania, USA. Later the next year in 1998, he became a fellow of the AAIP (American Academy of Implant Prosthodontics). Since 1994 Dr. Bassam had his private practice and since 1997 he has been limiting his dental work to Implants and Prosthodontics. In the year 2000, he owned the EIC (Esthetics Implant Center) which was one of the Implant specialized centers in Egypt, and since 2002, Dr. Bassam started his own private implant courses in addition to other implant courses he directed and was involved with some of the Elite Implant Associations inside and outside Egypt.

*Dr. Bassam Rabie* 

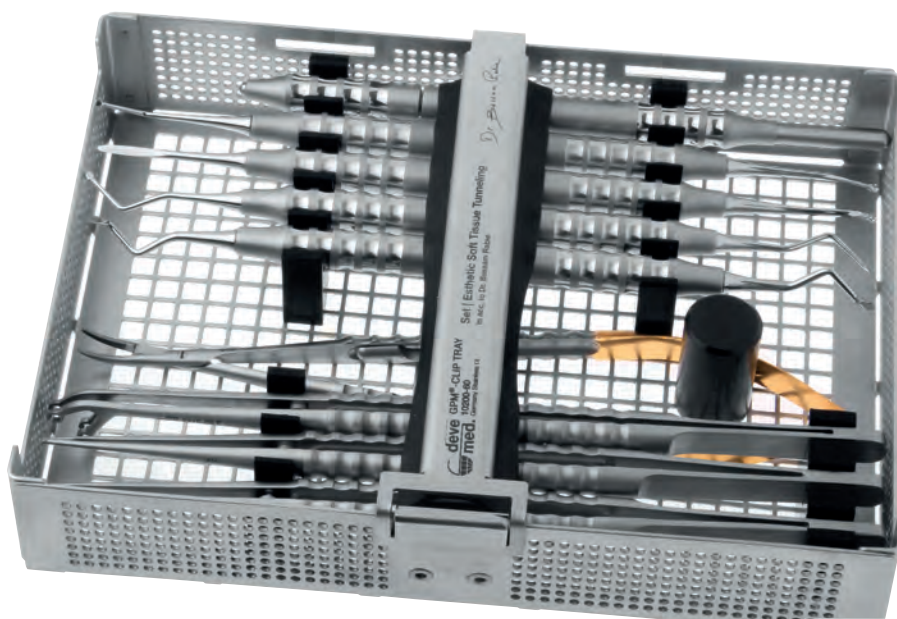
Dr. Bassam has lectured both nationally and internationally in many countries from international meetings to specialized implant courses specially in the topic of Biomechanical Considerations in Implant Prosthodontics, from a surgical and prosthodontic aspect and cosmetic and functional stability around implants in the esthetic zone, and Sinus Elevation and grafting techniques. Some of the countries are the United States of America, Spain, Italy, United Arab Emirates, Saudi Arabia, Libya and Syria. Dr. Bassam has demonstrated many live transmission surgeries in these implant courses. Dr. Bassam has published several articles in national and international magazines. Dr. Bassam now is a partner, chairman and medical director of the Presidential Esthetics Center & Institute which is rated the number one center in Egypt and one of the leading dental centers of the world.

PresiDental Esthetics  
13 Demashk st. Heliopolis  
Cairo  
Egypt

# ESTHETIC SOFT TISSUE TUNNELING SET nach Dr. Bassam Rabie

ESTHETIC SOFT TISSUE TUNNELING SET in accordance to Dr. Bassam Rabie

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The periosteal elevator was designed specifically for use in periodontal surgery. It is crucial for the success of the therapy, especially in relation to regenerative measures, that tight wound closure is achieved after the surgery. A prerequisite for this is the maximum possible preservation of soft tissue. This problem is presented in the interdental area. This very narrow and sharpened periosteal elevator simplifies this task.

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The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces.

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The mucous membrane transplant can be inserted into the pre-prepared tunnel using this instrument.

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PPS | Tissue graft plug, fine serrated

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Atraumatic micro forceps offer a compromise between the surgical forceps, which provide a strong grip yet damage tissue, and the less effective anatomical forceps, which however allow a gentle grip. The fine micro teeth prevent perforation and slippage of the tissue even at very slight pressure, thus enabling gentler gripping.

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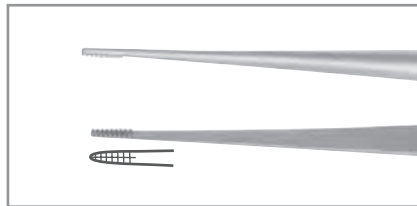
Surgical micro-forceps are equipped with interlocking teeth (1:2). The advantage of this is that soft tissue can be gripped very securely without pressing the forceps firmly together and greater tensile force can be applied. This prevents the tissue from being crushed. The disadvantage is that easily injured structures such as blood vessels and nerves can be destroyed by the sharp teeth. Anatomical forceps are more suitable when gripping structures of this kind.

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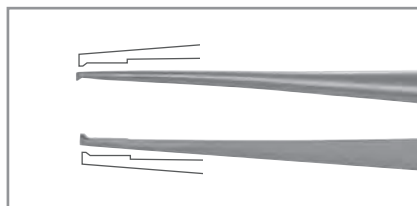
These suture tweezers allows an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously.

While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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This forceps serves perfect for fixation and suturing. The shaping of the double tweezers allows a safe reposition of the soft tissue graft. The circular tips avoid tearing of the tissue.

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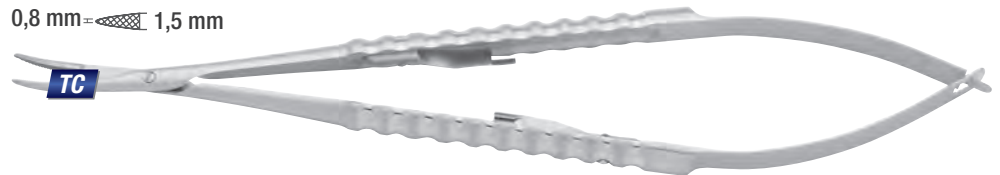
Micro needle holder with scissors and lock

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5-0 – 8-0

# IMPLANT GUIDED MICRO SURGERY SET nach Dr. Bassam Rabie

IMPLANT GUIDED MICRO SURGERY SET in accordance to Dr. Bassam Rabie

IMPLANT GUIDED MICRO SURGERY SET selon Dr. Bassam Rabie

Set: 10100-72

Artikel-Nr. Item no. Articles-Réf.	Bezeichnung Description Description	Set Artikel-Nr.: 10100-72 Item no. of the set: 10100-72 Set complet ref. 10100-72
1151-30 F 	Skalpelli Griff # 3 Scalpel handle # 3 Manche p. bistouri # 3	gerade, passend für 1 Klinge straight, suitable for 1 blade droit, pour 1 lame
1151-70 F 	Skalpelli Klingenhalter   doppelendig Scalpel blade holders   double ended Manches de bistouris   doubles	gerade, passend für 2 Klingen straight, suitable for 2 blades droit, pour 2 lames
2315-50 F 	Naht- und Membranpinzetten „TEAM-ATLANTA“ Suture and membrane forceps „TEAM-ATLANTA“ Pincettes à suture et à membrane „TEAM-ATLANTA“	17,3 cm, Ø 2,2 mm
2302-63 F 	Gewebefasspinzette „Cooley“ Atraumatic micro forceps „Cooley“ Micro-pinces à tissus „Cooley“	17,3 cm, 1,3 mm
999-10 F 	Abhalter „Emos“ Cheek „Emos“ Ecarteurs des joues „Emos“	17,3 cm

Artikel-Nr. Item no. Articles-Réf.	Bezeichnung Description Description	Set Artikel-Nr.: 10100-72 Item no. of the set: 10100-72 Set complet ref. 10100-72
<b>888-93 F</b>	 Raspatorien „Khoury“ Periosteal elevators „Khoury“ Rugines, décolleurs „Khoury“	17,3 cm
<b>888-B2 F</b>	 Mikro Raspatorium „Buser # 2“ Micro periosteal elevator „Buser # 2“ Micro-décolleur „Buser # 2“	17,3 cm
<b>901-24 F</b>	 Knochenküretten „Miller-Colburn # 9“ Bone curettes „Miller-Colburn # 9“ Curettes à os „Miller-Colburn # 9“	2,0/2,0 mm, 17,3 cm
<b>922-10 F</b>	 Sinus-Lift Elevator, # A1 Sinus-lift elevator, # A1 Sinus-Lift élévateur, # A1	17,3 cm
<b>921-10 F</b>	 Sinus-Lift Kürette # 1 „Palti“ Sinus-lift curette # 1 „Palti“ Sinus-lift curette # 1 „Palti“	4,5 mm / 4,5 mm, 17,3 cm
<b>923-60 F</b>	 Sinus-Lift Kürette „Tatum/Palti“ Sinus-lift curette „Tatum/Palti“ Sinus-lift curette „Tatum/Palti“	3,0 mm / 3,0 mm, 15 cm
<b>2406-90 F</b>	 Tiefenmeßgerät „Palti“ Depth gauge „Palti“ Jauge de profondeur „Palti“	graduiert graduated; graduation
<b>2500-35 F</b>	 Universal-Instrument „Biehler“ Universal instrument „Biehler“ Univesel instrument „Biehler“	17,3 cm, für Augmentationschirurgie 17,3 cm, for augmentation surgery 17,3 cm, pour chirurgie augmentation
<b>2500-81 F</b>	 Membran Platzierungsinstrument „Buser“ Membrane positioning instrument „Buser“ Instrument pour placement des membranes „Buser“	2,0 / 2,0 mm, 17,3 cm
<b>1170-65 F</b>	 Chirurgische Mikro-Schere „Gomel“ Operating micro-scissor „Gomel“ Micro-ciseaux chirurgicaux „Gomel“	16,0 cm, gebogen, fein gezahnt 16,0 cm, curved, fine serrated 16,0 cm, courbés, denture fine
<b>998-20</b>	Wangenabhalter Cheek retractor Ecarteur des joues	15,5 cm, „University of Cawood-Minnesota“ 15,5 cm, „University of Cawood-Minnesota“ 15,5 cm, „University of Cawood-Minnesota“
<b>1086-21 F</b>	 Minimal-invasiver Nadelhalter „Barraquer“ Minimal invasive needle holders „Barraquer“ Porte-aiguilles pour la chirurgie invasive minimisée „Barraquer“	gebogen, mit Kreuzriefung, 17,3 mm, Ø 1,2 mm curved, with cross serration, 17,3 mm, Ø 1,2 mm courbés, striure en forme de croix, 17,3 mm, Ø 1,2 mm
<b>2550-60</b>	Mischdose für Knochenmaterial Mixing bowl for bone material Cuvette de mélange pour mat. osseux	25 cc
<b>2406-50</b>	Mini Messlehre Mini-dental caliper Mini-jauge gradué	12,5 cm, 0,0-100,0 mm
<b>9100-00</b>	 GPM Wash Tray (275,0 x 180,0 x 35,0 mm)	mit Deckel   mit Haltegriff with top cover   with handhold avec couvercle   avec poignée

# IMPLANT GUIDED MICRO SURGERY SET nach Dr. Bassam Rabie

IMPLANT GUIDED MICRO SURGERY SET in accordance to Dr. Bassam Rabie

IMPLANT GUIDED MICRO SURGERY SET selon Dr. Bassam Rabie

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Scalpel blade holders | single-ended

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## Skalpellschliffhalter | doppelendig

Scalpel blade holders | double ended

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This retractor with a bend at both sides is used for specific surgery in defined regions.

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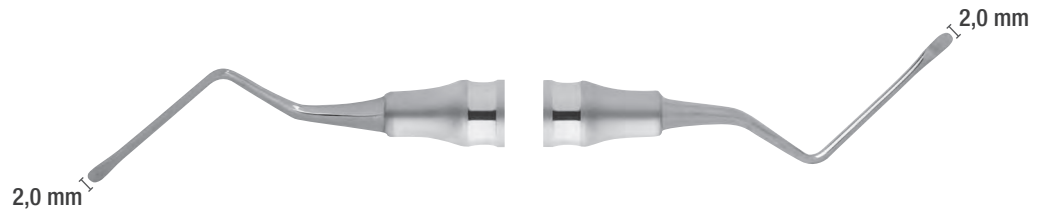
This depth gauge shown here allows you to accurately measure the depth of the implant cavity.

The neighbouring anatomical structures and the of the cavity can be optimally checked. The horizontal and vertical bone volume can also be precisely measured intra-operatively.

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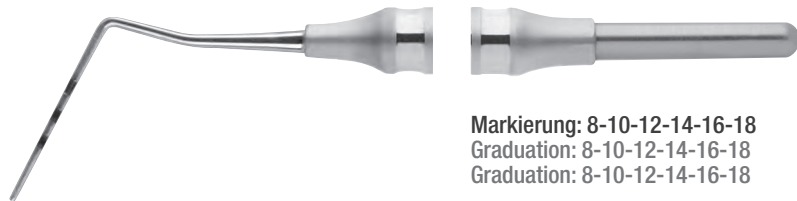
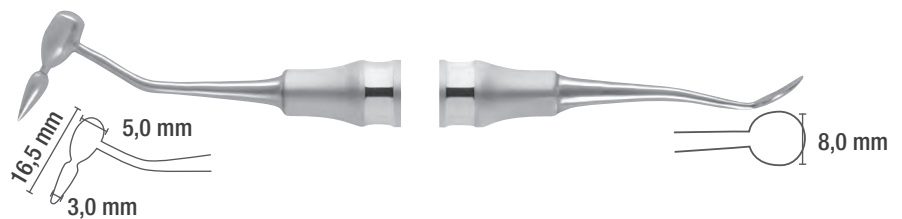
Scissors for minimalinvasive surgery

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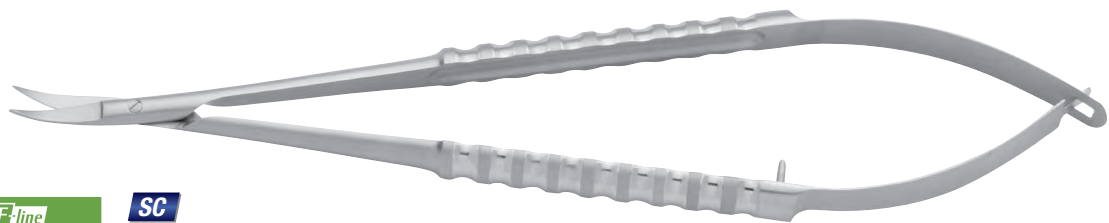
These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimal invasive surgery. As with the needle holders, the scissors also offer the possibility of a rotary movement. These scissors are ideally suited when working with magnifying glasses or a microscope.

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Minimal invasive needle holders with lock

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
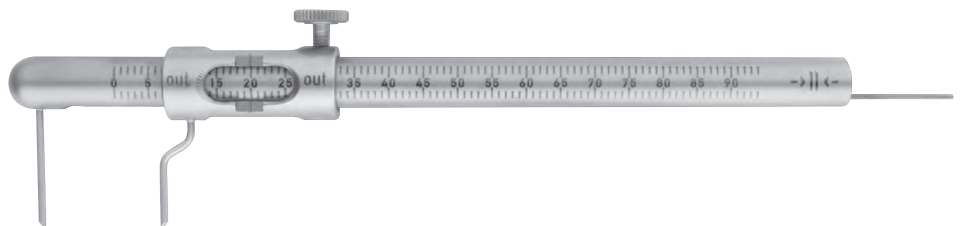
# BASIC GUIDED MICRO SURGERY SET nach Dr. Bassam Rabie

BASIC GUIDED MICRO SURGERY SET in accordance to Dr. Bassam Rabie

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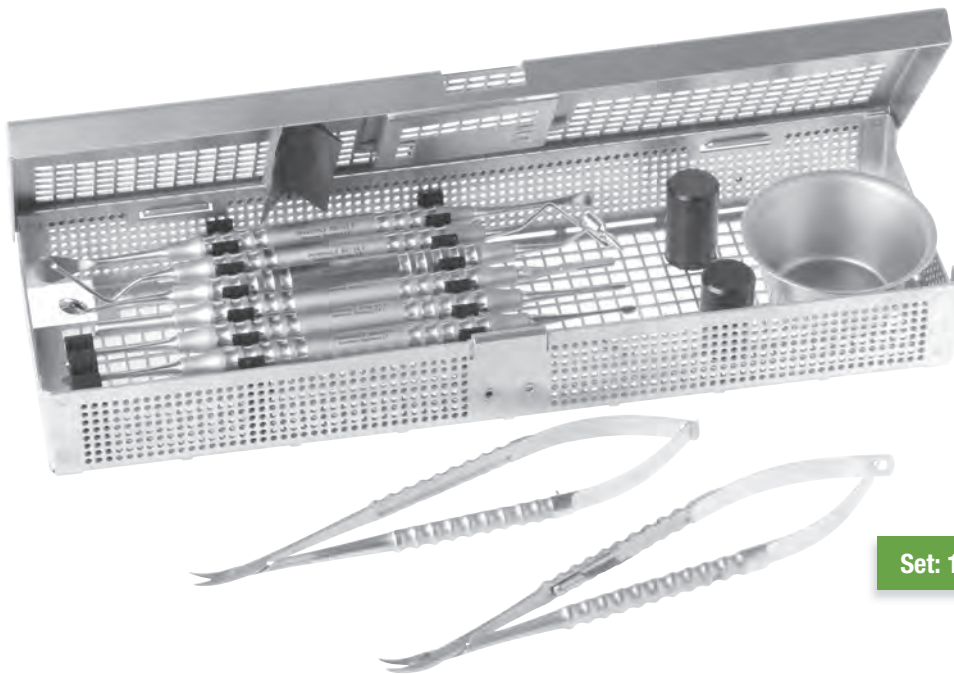The lance-shaped workpiece-end enables the detachment of the papilla as well as detachment of the marginal gingiva crest. The mucoperiosteal flap can then be gently detached with shovel-shaped workpiece-end on the opposite side.

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**devemed GmbH**