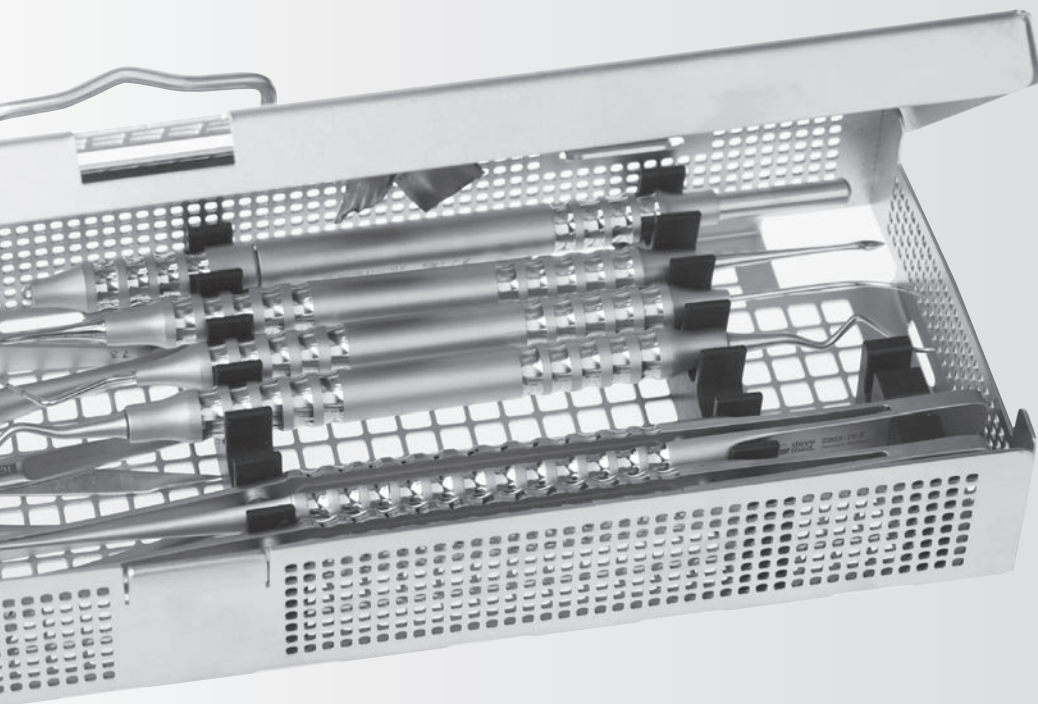
... makes the difference

**RECOMMENDATION OF
DR. IYAD GHONEIM**

Raspatorien

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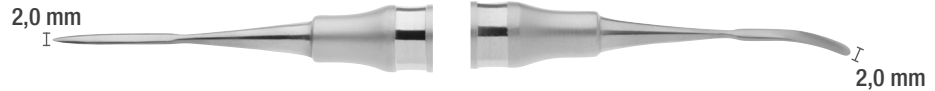
Starting at the Gingival mucosa, the attached mucous membrane can be surgically removed supra-periosteal with the slightly curved end of workpiece. The rounded tip prevents perforation of the mucosal flap. In the mobile mucosa area, the minimal invasive surgical removal can be continued with the straight, pointed end of workpiece.

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In comparison to PPS 1200 F this instrument differs considerably in the curvature of its workpieces. With a bridge flap, this instrument can be optimally used to prepare tunnels from apical to coronal through an entry in the periodontal membrane.

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The mucous membrane transplant can be inserted into the prepared tunnel using this instrument.

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These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimally invasive surgery. The scissors also offer the possibility of a rotary movement and are ideally suited for working with magnifying glasses or a microscope.

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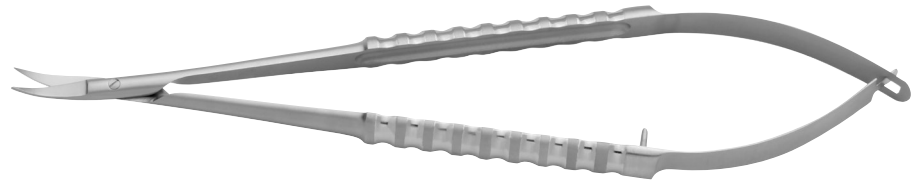
These suture forceps allows an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously.

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While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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Atraumatic micro-forceps offer a compromise between the surgical forceps, which provide a strong grip yet damage tissue, and the less effective anatomical forceps, which however allow a gentle grip. The fine micro teeth prevent perforation and slippage of the tissue even at very slight pressure, thus enabling gentler gripping.

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Anatomical micro-forceps can be recognized by fine cross grooves at the working ends. They are suitable for gentle gripping of more vulnerable structures such as blood vessels or nerves. The characteristic of being gentle on tissue does, however, have the disadvantage that the structures that have been gripped can slip from the forceps. That is why strong tensile force is not possible, in contrast to the surgical forceps.

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