... makes the difference

**RECOMMENDATION OF
DR. MARCELO FERRER BALART**

see it feel it test it

devemed
... makes the difference

Dr. Marcelo Ferrer

Dr. Marcelo Ferrer received his degree in Dentistry and his title of Dental Surgeon at University of Chile in 2000. After his postgraduate training at Universidad Andres Bello he received his degree of Master in Dentistry with specialization in Periodontics and Implantology.

With more than 17 years of teaching experience, he has been part of the faculty of the Prosthetics Dept. in the University of Chile, of the Adult Integrated Preclinical Department in the Universidad Mayor and of the Faculty of Dentistry, University Andres Bello, in the area of Periodontics and Implantology, both in Pre and Postgraduate courses since 2003.

Currently, he exercises in his private practice and as National Director of the Postgraduate Course in Periodontics and Surgical Implantology at San Sebastian University. National and International Speaker for more than 10 years. Director of South Cone Implantology Training Center Chile and Dental Xp Expert for some years.

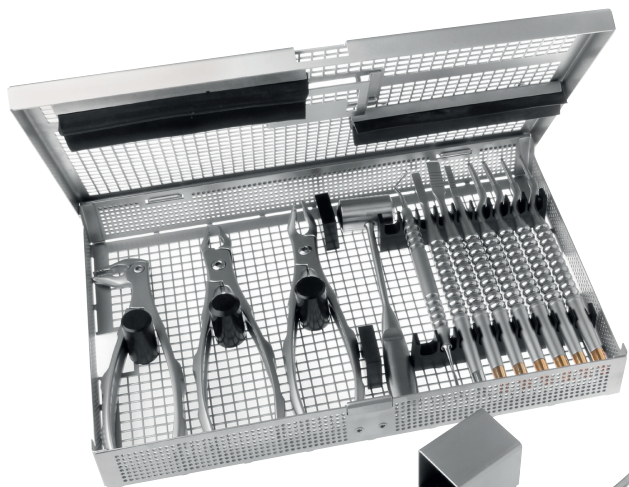
Dr. Ferrer is an active member of the American Academy of Osseointegration, member of the Society of Periodontology and of the Society for Prosthetics and Oral Rehabilitation of Chile.

SIMPLIFIED EXTRACTION SET nach Dr. Marcelo Ferrer Balart

SIMPLIFIED EXTRACTION SET in accordance to Dr. Marcelo Ferrer Balart

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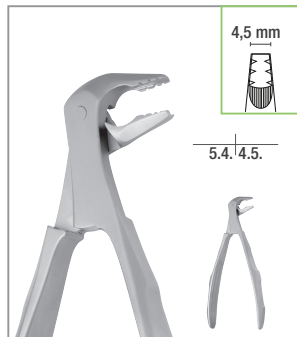
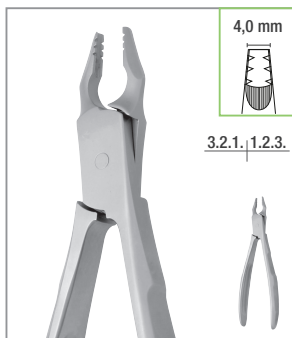
The sharp workpiece end can be used to optimally open and stretch the alveolar ridge, after the use of a diamond cutting wheel. The 2 mm graduations ensure precise monitoring of the penetration depth.

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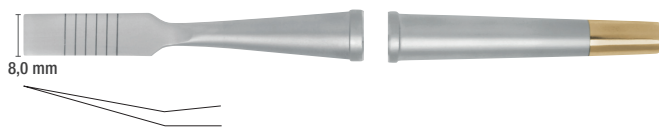
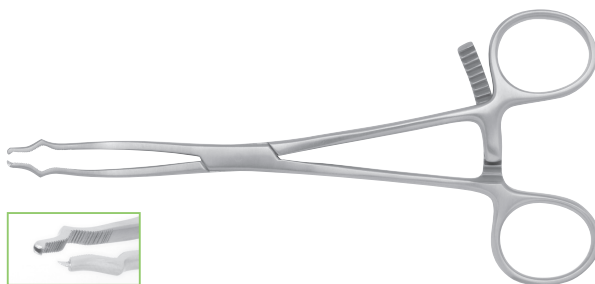
These bone holding forceps is especially suitable for fixing a bone block on the jaw defect. The bone can be predrilled and held for screwing firmly onto the defect without changing grip. The serration at the tip gives the bone block extra hold.

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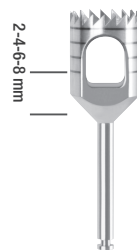
These trephine burrs are appropriate for all angular pieces.

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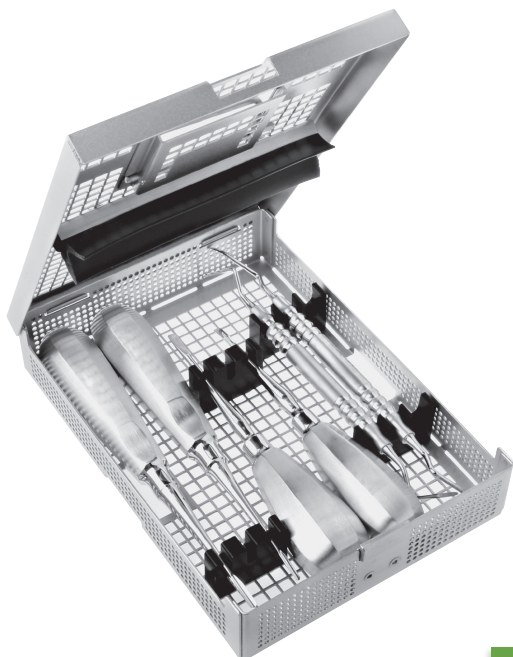
Rack for trephine burrs up to a length of 35,0 mm

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LUXATION SET in accordance to Dr. Marcelo Ferrer Balart

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TUNNELING SET in accordance to Dr. Marcelo Ferrer Balart

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The lance-shaped end of the workpiece enables the detachment of the papilla as well as of the marginal gingiva crest. The mucoperiosteal flap can then be gently detached with the shovel-shaped end of the workpiece on the opposite.

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Starting at the Gingival mucosa, the attached mucous membrane can be surgically removed supra-periosteal with the slightly curved end of workpiece. The rounded tip prevents perforation of the mucosal flap. In the mobile mucosa area, the minimal invasive surgical removal can be continued with the straight, pointed end of workpiece.

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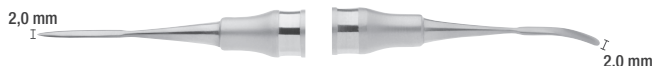
The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces.

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The mucosal flap preparer is particularly suitable for the posterior tooth area. Access and atraumatic mucosal flap preparation is possible due to the slightly curved workpieces.

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The mucous membrane transplant can be inserted into the prepared tunnel using this instrument.

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PPS | Tissue graft plug, fine serrated

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
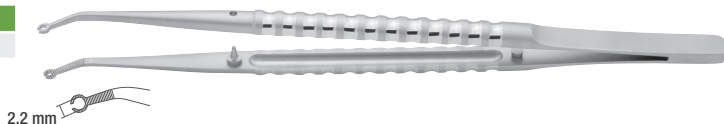
17,3 cm

BONE GRAFT & TISSUE SET nach Dr. Marcelo Ferrer Balart

BONE GRAFT & TISSUE SET in accordance with Dr. Marcelo Ferrer Balart

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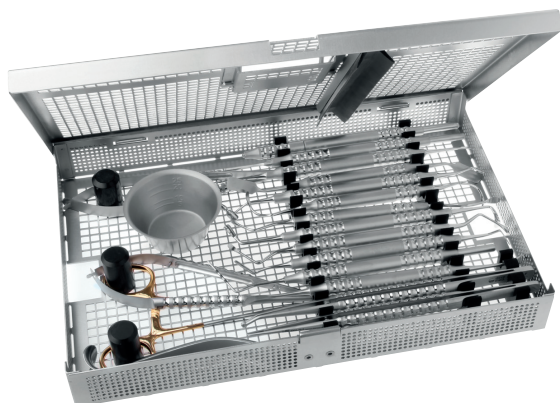Micro scalpel blade holders | single-ended

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The screw cap allows a simple and safe insertion and removal of the standard scalpel blade.

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Light, rotatable and handy for precise circular incisions.

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Scalpel blade holders | single-ended

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Anatomical micro-forceps can be recognized by fine cross grooves at the working ends. They are suitable for gentle gripping of more vulnerable structures such as blood vessels or nerves. The characteristic of being gentle on tissue does, however, have the disadvantage that the structures that have been gripped can slip from the forceps. That is why strong tensile force is not possible, in contrast to the surgical forceps.

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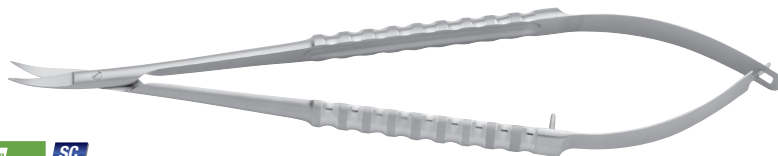
These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimally invasive surgery. The scissors also offer the possibility of a rotary movement and are ideally suited for working with magnifying glasses or a microscope.

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gebogen
curved
courbés

BONE GRAFT & TISSUE SET nach Dr. Marcelo Ferrer Balart

BONE GRAFT & TISSUE SET in accordance to Dr. Marcelo Ferrer Balart

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These depth gauges allow to accurately measure the depth of the implant cavity.

The neighbouring anatomical structures and the integrity of the cavity can optimally be checked. The horizontal and vertical bone volume can also be precisely measured intraoperatively.

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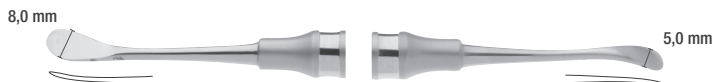
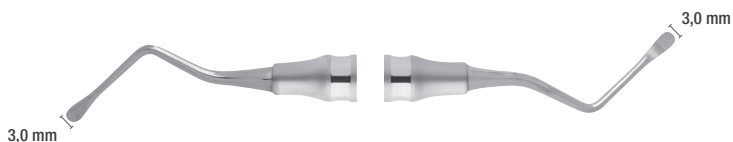
Mazor # 2 17,3 cm

BONE GRAFT & TISSUE SET nach Dr. Marcelo Ferrer Balart

BONE GRAFT & TISSUE SET in accordance to Dr. Marcelo Ferrer Balart
BONE GRAFT & TISSUE SET selon Dr. Marcelo Ferrer Balart

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This retractor with a bend at both sides is used for specific surgery in defined regions.

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These suture forceps allow an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously.

While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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devemed GmbH