PERIOLUXE

PERIOLUX
PÉRIOLUXE

SWE | PERIOLUXE

SWE | PERIOLUX
SWE | PÉRIOLUXE

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Nowadays the modern implantology takes center stage with the immediate-implantation after gentle and atraumatic tooth extraction. Therefore it is more and more important to treat the hard and soft tissue compositions very gentle and atraumatic, already during the tooth extraction.

The perioluxes are a further development of the periotomes and enable a unique mesial and distal manipulation of the structure of the root. This prevents a fracture of the buccal and labial bone lamella, which is extremely important for the implant insertion. The perioluxes are an ideal completion to the conventional extracting instruments for a gentle extraction of non-preservable teeth.

- Light handle design for fatigue-proof working
- The size of the handle allows a careful and controlled working as well as a direct transmission
- Atraumatic removing of teeth without trauma of the nearby structure

The alveole is expanded atraumatically through the especially flat working ends and the periodontal ligaments are carefully severed through a light rotating circular motion.

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
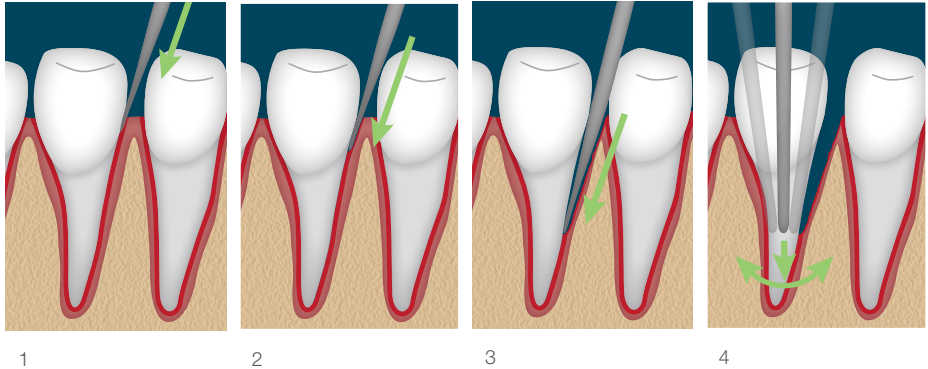
Attach the PeriLux to the tooth (1), enlarge the alveole (2), sever the ligament (3) and ease the tooth with a rotating circular motion of its socket (4). The result is a minimum damage of tissue

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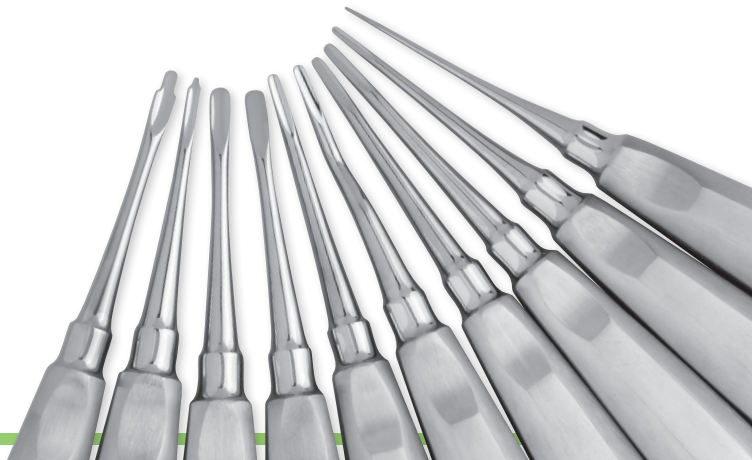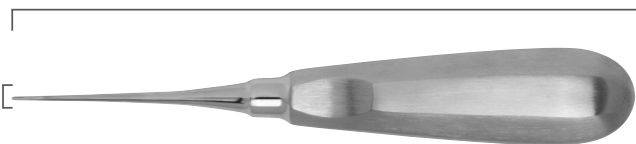
The TIN Perioluxes can be easily inserted along the root surface and cut the periodontal ligament prior to extraction (Caution: no levering). They feature a long, slim profile and sharp blades, minimising tissue trauma and preserving the alveolar bone. The TiN coating improves gliding in the periodontal gap due to its lower coefficient of friction and improves the durability of the blades.

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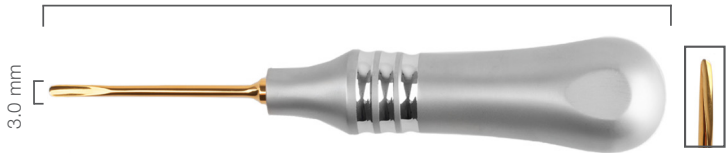
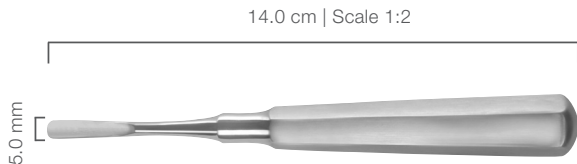
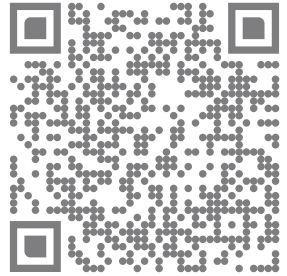
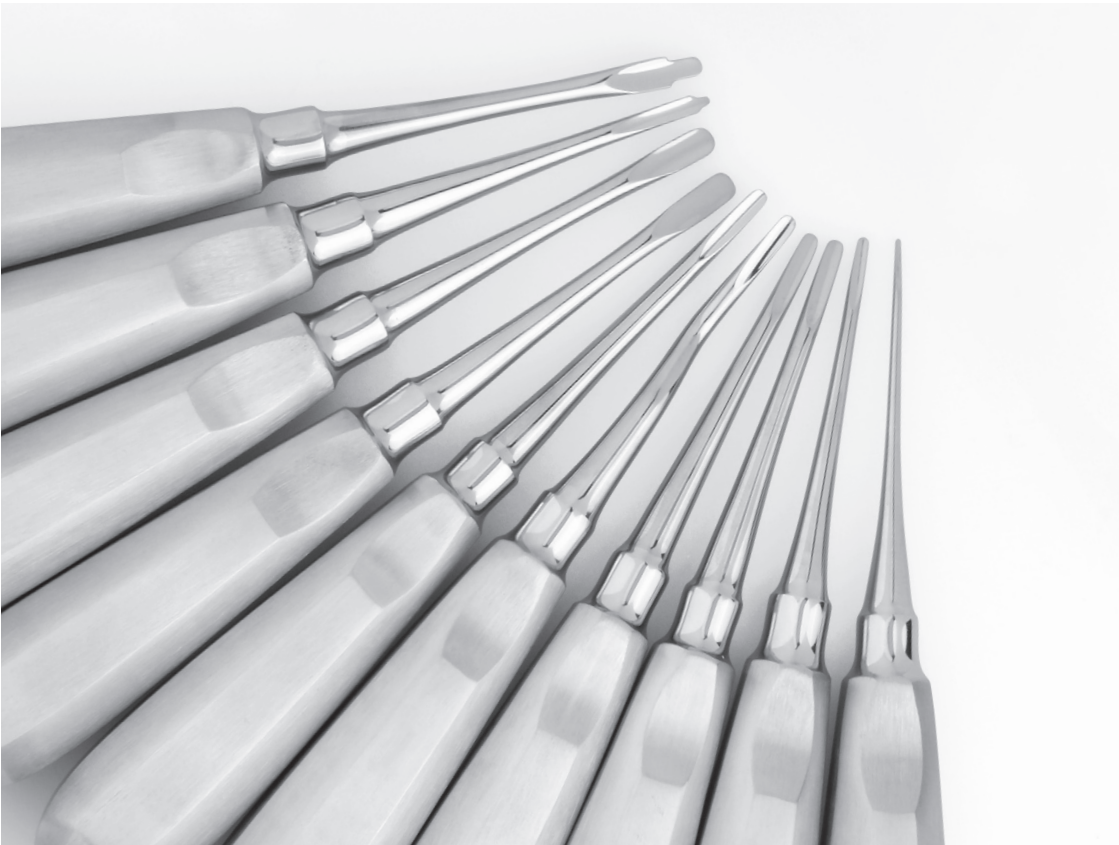
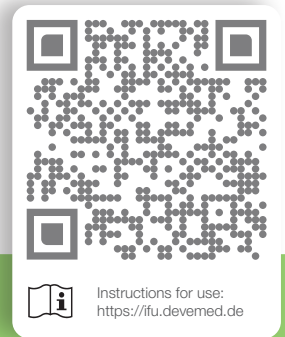
All listed medical devices bear the CE mark. Products above Class I also bear the identification number 0483 of our notified body. Medical devices that we merely distribute are marked accordingly on the relevant pages.

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devemed GmbH