RECOMMENDATION OF
DR. ANA LUISA BERNOTTI

DR. ANA LUISA BERNOTTI

DDS. Periodontist – Implantologist.

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- Periodontist 1999. Universidad Central de Venezuela
- Implantologist 2015. Centro de especialidades y Estudios
- Implantologist 2015 . CEYESOV . Mexico.
- Active Member of Periodontology Society of Venezuela. Since 2000.
- Active Member of Iberoamerican Federation of Periodontology. Since 2000.
- Author of "Bone wedge technique" 2017.
- National and International Speaker.
- Private Practice: Bernotti Odontologia Avanzada[®]. Caracas - Venezuela.
- Active member "Follow The Pink" & "Divas in Dentistry" website

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The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels. The spatula-shaped working part can be used to attach bone material etc. and at the same time to retract the gingiva.

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The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces. To avoid injury to the gingiva, the side of the working part which faces the gum is blunt.

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The lance-shaped end of the workpiece enables the detachment of the papilla as well as of the marginal gingiva crest. With the slightly curved working part the gingiva is suprapariostally removed/loosened - the rounding of the working part prevents the mucosal flap from being pierced.

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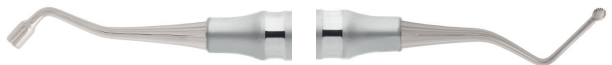
The tissue graft plug is used to insert the mucosal graft or membrane. The granulate pluggler with serration can be used to compact bone material.

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
Bernotti | 17,1 cm

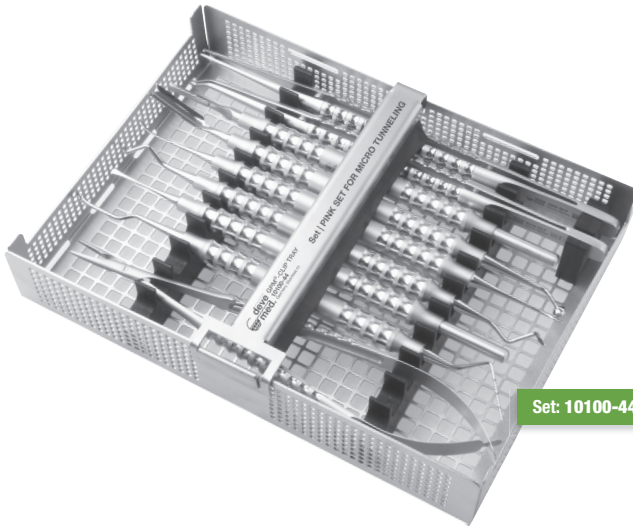PINK SET FOR MICRO TUNNELING nach Dr. Ana Luisa Bernotti

PINK SET FOR MICRO TUNNELING in accordance to Dr. Ana Luisa Bernotti

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The gingivectomy knife can be used as a micro-blade for the tunnel technique due to its flexible, sharp working end.

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Scalpel blade holders | single-ended

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Tissue graft plug, fine serrated | Granulate plugger

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The tissue graft plug is used to insert the mucosal graft or membrane. The granulate plugger with serration can be used to compact bone material.

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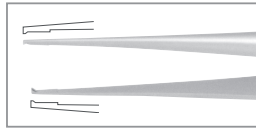
These suture forceps allows an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously.

While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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devedmed GmbH