RECOMMENDATION OF
DR. HENRY HO

Dr. Henry Ho – Curriculum Vitae

Dr. Henry Ho graduated from the National University of Singapore in 1996 and received his postgraduate degree (Prosthodontics) in 2002. He is a member of the Royal College of Surgeons (Edinburg) and specializes in prosthodontics and implant dentistry.

Dr. Ho has a keen interest in education and lectures internationally. In Singapore, he is a visiting specialist at the National University Hospital and adjunct lecturer at the NUS.

Dr. Henry Ho is a board member of the dental implant postgraduate program and also a regional trainer for the University of Frankfurt.

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The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels.

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The gingivectomy knife can be used as a micro-blade for the tunnel technique due to its flexible, sharp working end.

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2,0 mm

IMPLANT AND SURGERY SET nach Dr. Henry Ho

IMPLANT AND SURGERY SET in accordance to Dr. Henry Ho

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Minimal invasive needle holders with lock

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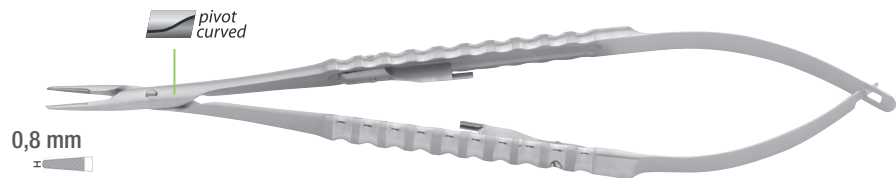
5-0 – 8-0

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Scalpel blade holders | single-ended

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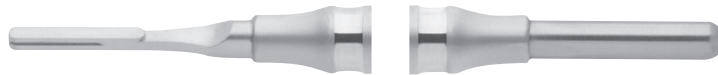
Scissors for minimal invasive surgery

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These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimal invasive surgery. As with the needle holders, the scissors also offer the possibility of a rotary movement. These scissors are ideally suited when working with magnifying glasses or a microscope.

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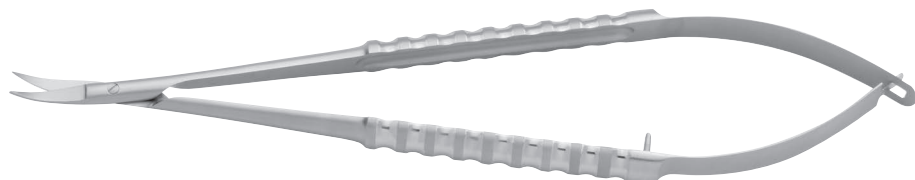
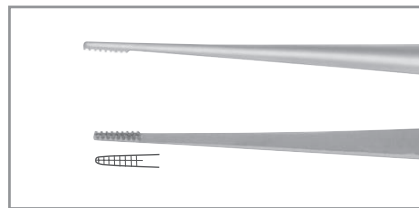
Atraumatic micro-forceps offer a compromise between the surgical forceps, which provide a strong grip yet damage tissue, and the less effective anatomical forceps, which however allow a gentle grip. The fine micro teeth prevent perforation and slippage of the tissue even at very slight pressure, thus enabling gentler gripping.

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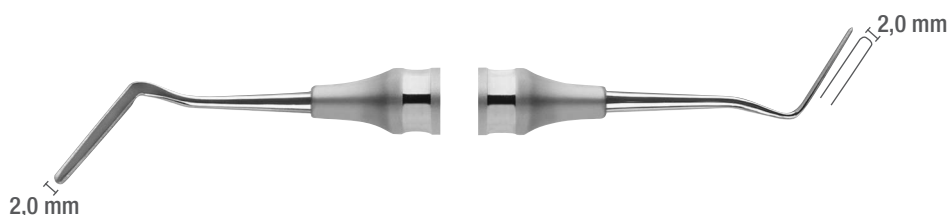
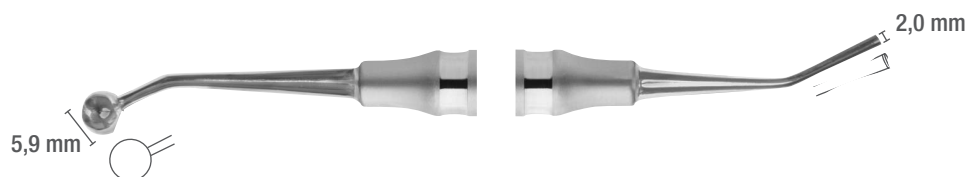
This applicator ensures the optimal blending and transportation of bone grafting material. The narrow workpiece is available for smaller and the wider workpiece for larger quantities.

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devemed GmbH