RECOMMENDATION OF
DR. ROLAND TÖRÖK

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ResearcherID O-2864-2013

Dr. Roland Török is the founder of „Implant Institute Török“, which was developed in 2008 to provide quality continuing education on the subject of dental implants and bone management. He is organizing meetings and live surgery courses in his institute.

Since 2003 he is lecturing in Germany and many other countries worldwide (Austria, Poland, Slovakia, Swiss, Romania, Croatia, Serbia, Bulgaria, Slovenia, Makedonia, Portugal, Italy, France, Israel, Taiwan, Dubai, Hungary, Turkey, Cyprus, United States, Russia, Thailand, Vietnam, China...) about implants and bone regeneration.

At the same time, Dr. Török is working in the development for augmentation materials and implant design with different companies.

DMD Török is dentist and dental technician, maintains a private practice for general dentistry with focus on implants and prosthodontics in Nuernberg/Germany, and he was the youngest specialist in the “Who’s who“ of Implantology.

He actually teaches Implantology in the Curriculum Implantology ICOI Europe (Germany), ICOI China, PSI (Polish society for Implantology) and taught already in the University „Carol Davila“ in Bucharest/Romania in the department for Maxillo Facial Surgery. Dr. Török published as well articles about augmentation and Sinus-Lift, and he serves on the editorial board in the Journal for Maxillo Facial Surgery (Romania) and POSEIDO.

Active member in numerous societies such as:

- President ICOI Europe for Germany
- Ambassador ICOI
- Diplomate ICOI
- BDIZ
- EDI (The European Association of Dental Implantologists)
- SRCOMF (Romanian Association for Oral and Maxillo Facial Surgery),
- Program coordinator and editor „Poseido“
- Councilor of the SEUCC (south European Union counter cancer)/ SEHNOS (south European head and neck oncology society).

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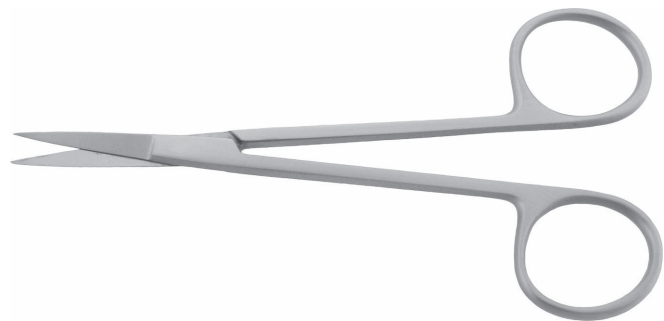
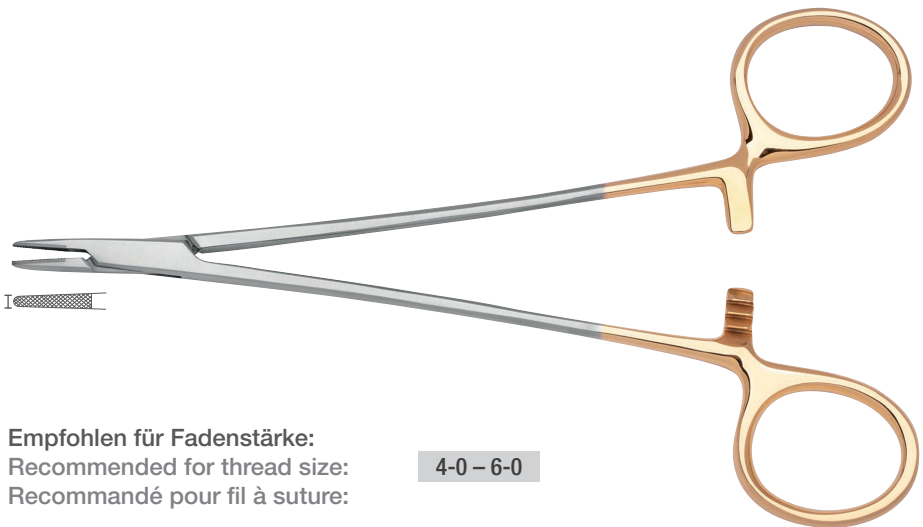
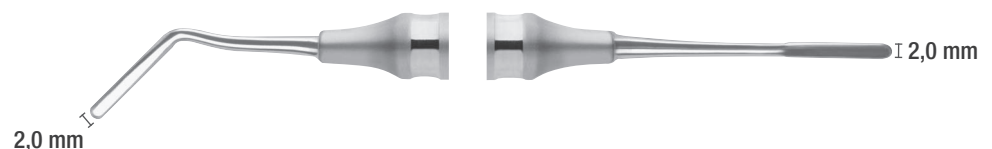
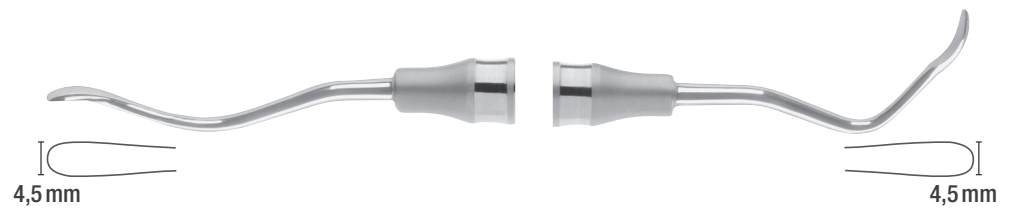
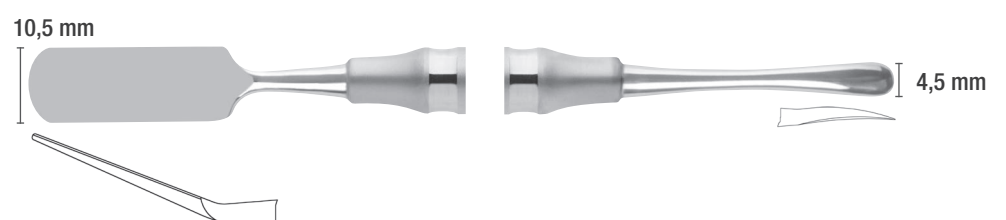
This applicator ensures the optimal blending and transportation of bone grafting material. The narrow workpiece is available for smaller and the wider workpiece for larger quantities.

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These depth gauges allow to accurately measure the depth of the implant cavity.

The neighbouring anatomical structures and the integrity of the cavity can optimally be checked. The horizontal and vertical bone volume can also be precisely measured intraoperatively.

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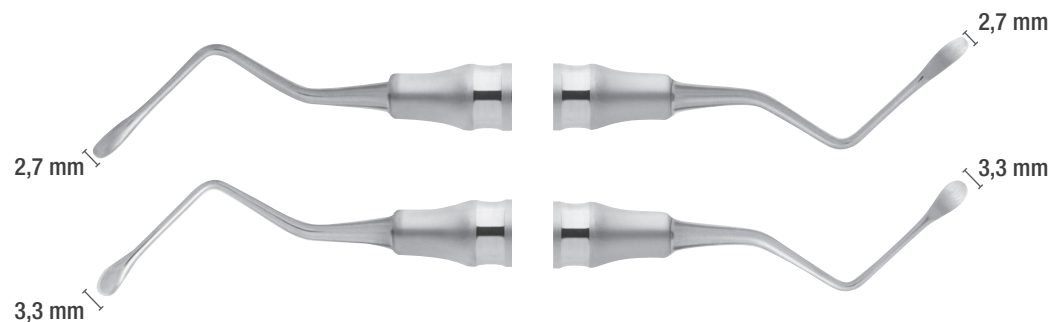
OD = outer diameter
ID = inner diameter

CHIRURGIE SET nach Dr. Roland Török

SURGERY SET in accordance to Dr. Roland Török

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Atraumatic micro-forceps offer a compromise between the surgical forceps, which provide a strong grip yet damage tissue, and the less effective anatomical forceps, which however allow a gentle grip. The fine micro teeth prevent perforation and slippage of the tissue even at very slight pressure, thus enabling gentler gripping.

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Anatomical micro-forceps can be recognized by fine cross grooves at the working ends. They are suitable for gentle gripping of more vulnerable structures such as blood vessels or nerves. The characteristic of being gentle on tissue does, however, have the disadvantage that the structures that have been gripped can slip from the forceps. That is why strong tensile force is not possible, in contrast to the surgical forceps.

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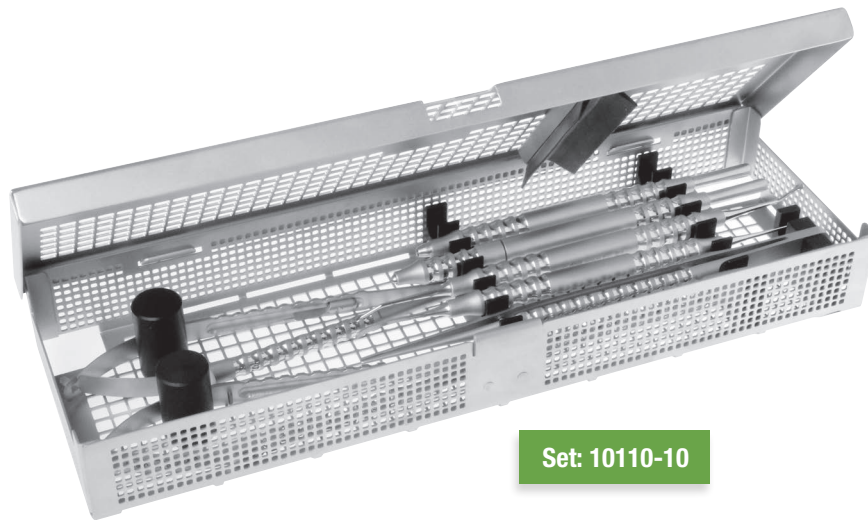
The screw cap allows a simple and safe insertion and removal of the standard scalpel blade.

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Micro needle holders with lock for minimally invasive surgery

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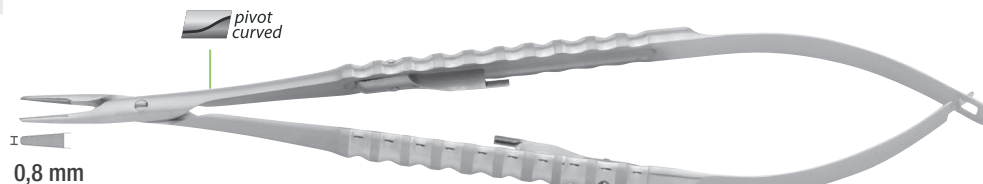
Scissors for minimally invasive surgery

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These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimally invasive surgery. The scissors also offer the possibility of a rotary movement and are ideally suited for working with magnifying glasses or a microscope.

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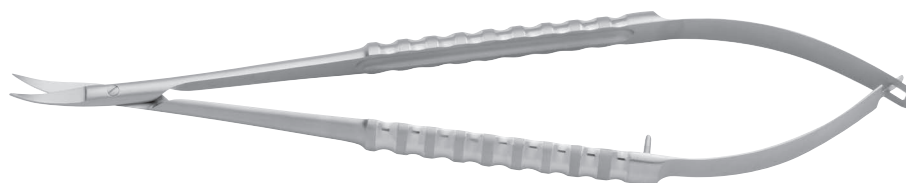
gebogen
curved
courbés

OSTEOSYNTHESE SORTIMENT

OSTEOSYNTHESIS ASSORTMENT OSTEOSYNTHESIS ASSORTIMENT

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The screw head with the slightly conical Quadro-Profile prevents the screw from spinning and enables better purchase on the screwdriver. This ensures secure insertion and placement.

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
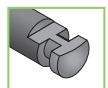
Screwdriver blades are available in two lengths. The QUADRO-Profile prevents the screw from spinning and enables better purchase. Only one blade (black) is required for the 1.3 mm, 1.5 mm and 1.8 mm screw systems.

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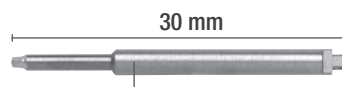
OSTEOSYNTHESE SORTIMENT


OSTEOSYNTHESIS ASSORTMENT
OSTEOSYNTHESIS ASSORTIMENT

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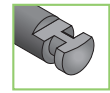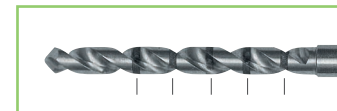The TITANIUM pins are used to secure titanium meshes and membranes. HEX HEAD pins can be easily removed after healing using the pin driver due to the thread.

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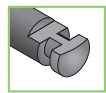
The applicator facilitates convenient insertion of the pins from the toolbox and direct application in the jaw. With its dental attachment, the applicator is suitable for the dev-os manual turning handle.

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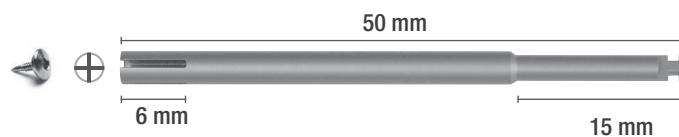
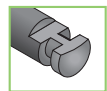
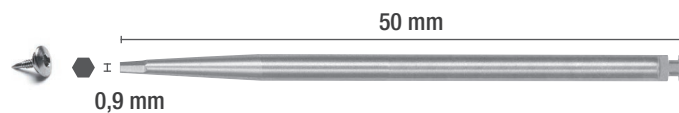
The driver allows the HEX HEAD pins inserted in the jaw to be quickly and easily removed after healing.

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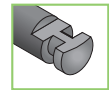
- Suitable for
 - Screwdriver blades
 - Pin applicator
 - Pin driver
- Convenient handling
- Fixed pressure point at the rear, silicone handle can be rotated
 - > Simplified tightening and loosening of screws and pins

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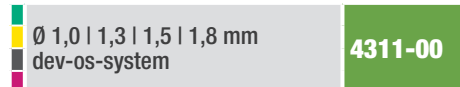
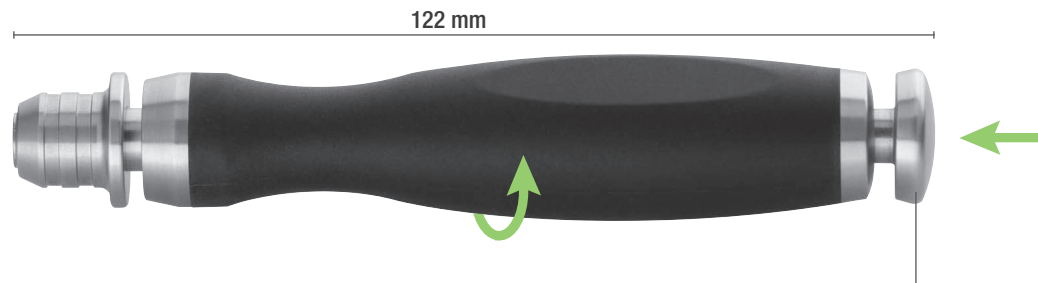
Place the pressure point against the palm of your hand and turn the silicone handle using your fingers. This enables simple and reliable turning by hand.

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- Clear arrangement (quick correlation)
- Individually configurable
- Compartments for:
 - Handle
 - Screws, in four different diameters and up to six lengths
 - Pins, in two different lengths
 - Various drills
 - Screwdriver blades (short/long)
 - Pin applicator
 - Pin driver
- Suitable for instrument reprocessing (ultrasonic bath, thermal disinfection, sterilization)
- Supports simple, direct storage of screws and pins
- Measurement guide for measuring screws, pins and drills

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devemed GmbH